



Quick Reference Guide

Anal Cytology Specimen Collection

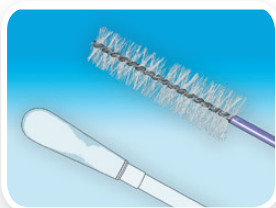

Collect...

adequate sample using a moistened Dacron[®]* swab or cytobrush¹ following published collection techniques².

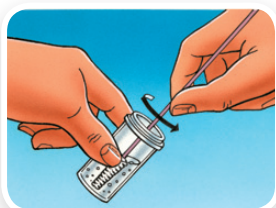

Rinse...

swab/brush as quickly as possible in the PreservCyt[®] Solution by rotating the device in the solution 10 times while pushing against the PreservCyt vial wall. Swirl the device vigorously to further release material. Discard the device.



Tighten...

the cap so that the torque line on the cap passes the torque line on the vial.



Record...

the patient's name and ID number on the vial.

Record...

the patient information and medical history on the cytology requisition form.



Place...

the vial and requisition in a specimen bag for transport to the laboratory.

1 Darragh, TM, Birdsong, GG, Anal Rectal Cytology, The Bethesda System for Reporting Cervical Cytology, 2004, 2nd Edition, 169-175

2 Bean, SM, Chhieng, DC, Anal-Rectal Cytology: A Review. Diagnostic Cytopathology 2009; Vol 38 No 7, 538-546

*Dacron[®] is a registered trademark of Invista